

## OAKMONT AEDs

There are 5 registered AEDs in Oakmont.

- **TYPE: “ZOLL AED Plus”**

Each AED comes with manufacturer instructions and supporting documentation for use and maintenance. There is a routine maintenance schedule in place, and the AEDs are registered. Because they are registered, they can also be located through apps such as PulsePoint and the AED Registry, which allow users to see nearby AED locations in an emergency.

- **APPS TO TRACK AEDs**



**PulsePoint Respond**

Building informed communities





**PulsePoint AED**

- **REGISTERED OAKMONT AED LOCATIONS (on “PulsePoint AED”)**


1. **Oakmont West Rec (WRC)**
  - Inside WRC, 2nd floor, south wall
2. **Berger Center**
  - Inside Berger Center next to Kitchen
3. **OVA Office**
  - OVA office hall
4. **Oakmont CAC**
  - Inside CAC left hallway near library
5. **Oakmont East Rec (ERC)**
  - Inside ERC, to right of main entrance

The “PulsePoint AED” app on your phone shows the locations of the AEDs on a map. Examples below are the OVA Office and Berger Center.

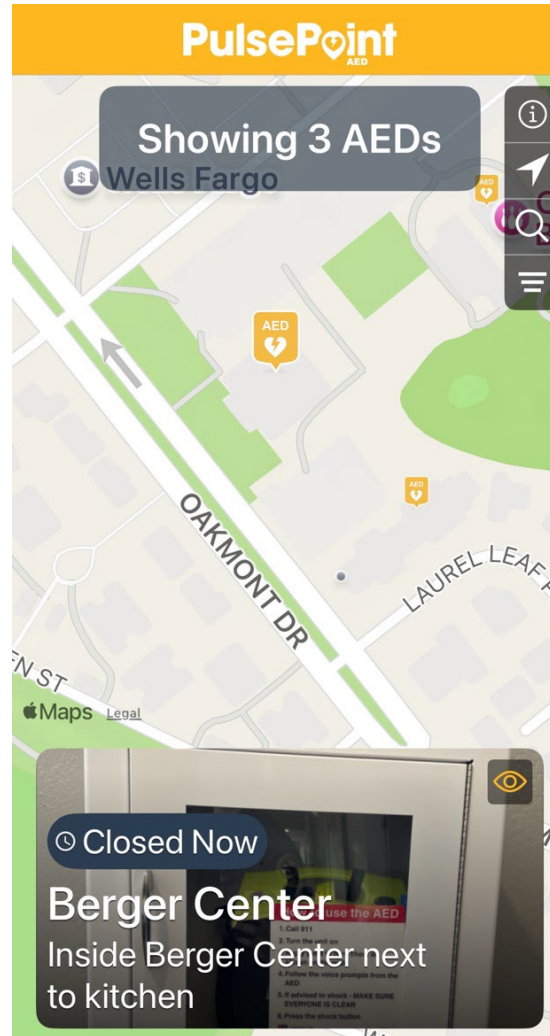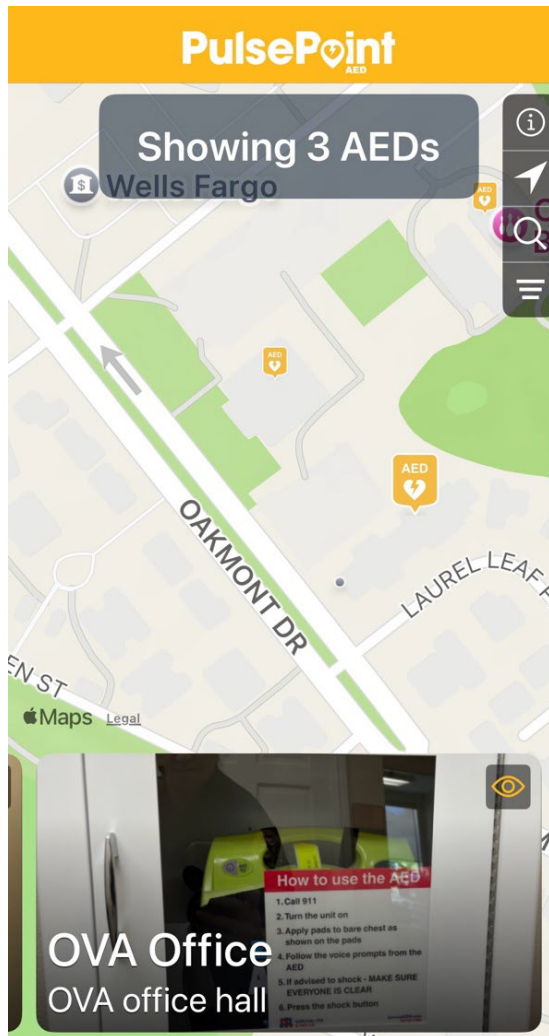

# ZOLL AED Plus – Emergency Preparedness Guide

## AED Cabinet (ZOLL AED Plus)

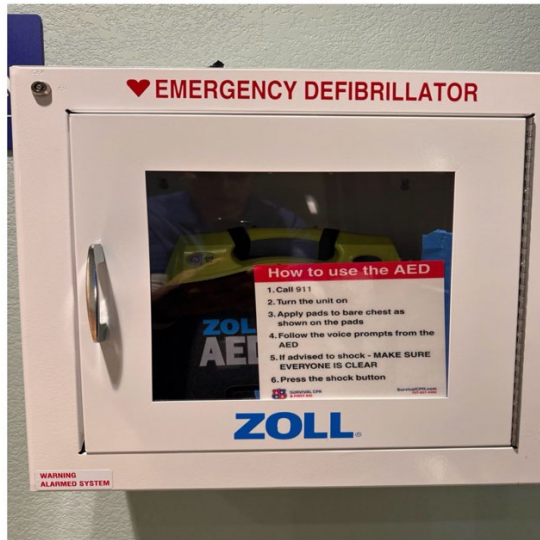

### How to Use the AED

- Call 911 immediately.
- Turn the AED ON.
- Expose bare chest and attach CPR-D-padz®.
- Follow voice prompts.
- Ensure everyone is CLEAR before shock.
- Resume CPR immediately after shock.

### Post-Incident Procedure

- Continue care until EMS arrives.
- Notify leadership and complete incident report.
- Remove AED from service and label 'Out of Service'.
- Replace pads and batteries.
- Verify green readiness indicator before returning to service.
- Conduct debriefs within 24–72 hours.

### Monthly Readiness Checklist

- Green check visible
- Pads not expired
- Batteries not expired
- Cabinet alarm functional
- AED registered correctly in PulsePoint



# PulsePoint AED

